



JOBST® Anti-Em/GP™

Designed to reduce
the risk of deep vein
thrombosis in the
hospital patient

Seamless elastic stockings with gradient compression



Comfort, Health and Style!

JOBST® Anti-Em/GP™

Designed for the hospitalized patient

In cooperation with leading vascular specialists, JOBST engineers developed the JOBST Gradient Compression Seamless Anti-Embolism Elastic Stocking specifically designed to combat thromboembolism. The JOBST Anti-Em/GP Stocking does this by using gradient compression. Gradient compression requires that the greatest amount of pressure be applied to the ankle with this pressure gradually lessening up the length of the leg. Gradient compression helps improve the ability of the veins to move blood back toward the heart.

The use of JOBST Anti-Em/GP Seamless Stockings is an accepted prophylaxis for minimizing the risk of thromboembolism in hospitalized patients requiring bed rest or prolonged periods of immobilization. Three styles are available: Knee High, Thigh High, and Waist High.

Features & Benefits

- Accurate gradient compression therapy minimizes the risk of DVT (blood clots).
- Soft, breathable fabric is lightweight and comfortable.
- Reinforced heel makes stockings more durable.
- Latex free formula helps avoid allergic reactions.
- An opening on the top of the foot allows for easy visual inspection of the foot and toes.
- The number of sizes ensures a proper fit which increases patient compliance.

Styles

Knee High



Thigh High



Knee High, Thigh High, and Waist High are available for both Men and Women.
(Waist High not pictured)

Color

White



Care instructions


- Machine wash in lukewarm water with delicate fabric detergent.
- Tumble dry on low heat.
- Do not bleach, dry clean or iron.

Fiber contents

- Knee High: 85% Nylon; 15% Spandex
- Thigh High: Small – X-Large: 79% Nylon; 17% Spandex, 4% Neoprene
XX-Large: 77% Nylon, 19% Spandex, 4% Neoprene
- Waist High: 75% Nylon; 25% Spandex





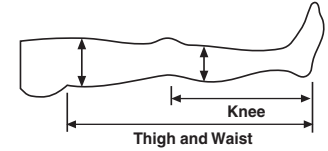









Measuring and Ordering Information

- Knee: Measure calf circumference and length from heel to bend of knee.
- Thigh and waist: Measure calf and thigh circumferences, and length from heel to gluteal fold.
























Toe color matches color-coded sizing on each polybag and with sizing chart.













KNEE HIGH

Size	Calf Circumference 	Length (Toe Color) 	Catalog #
S 	9"-12" 23-30 cm	Small Regular  14" - 17" 36-43 cm	111402
		Small Long  17" - 20" 43-51 cm	111403
M 	12"-15" 30-38 cm	Medium Regular  14" - 17" 36-43 cm	111406
		Medium Long  17" - 20" 43-51 cm	111407
L 	15"-17½" 38-44 cm	Large Regular  14" - 17" 36-43 cm	111410
		Large Long  17" - 20" 43-51 cm	111411
X-L 	17½"-20" 44-51 cm	X-Large Regular  14" - 17" 36-43 cm	111414
		X-Large Long  17" - 20" 43-51 cm	111415

THIGH HIGH

Size	Calf Circumference 	Thigh Circumference 	Length (Toe Color) 	Catalog #
S 	9"-12" 23-30 cm	16"-29" 41-74 cm	Small Short  25" - 29" 64-74 cm	111450
			Small Regular  29" - 33" 74-84 cm	111451
			Small Long  33" - 38" 84-97 cm	111452
M 	12"-15" 30-38 cm	16"-29" 41-74 cm	Medium Short  25" - 29" 64-74 cm	111454
			Medium Regular  29" - 33" 74-84 cm	111455
			Medium Long  33" - 38" 84-97 cm	111456
L 	15"-18" 38-46 cm	16"-29" 41-74 cm	Large Short  25" - 29" 64-74 cm	111458
			Large Regular  29" - 33" 74-84 cm	111459
			Large Long  33" - 38" 84-97 cm	111460
X-L 	18"-20" 46-51 cm	24"-31" 61-79 cm	X-Large Regular  29" - 33" 74-84 cm	111462
			X-Large Long  33" - 38" 84-97 cm	111463
XX-L 	18"-21" 46-53 cm	32"-36" 81-90 cm	XX-Large Regular  25" - 32" 64-81 cm	111464
			XX-Large Long  32" - 38" 81-97 cm	111465

WAIST HIGH

Size	Calf Circumference 	Thigh Circumference 	Length 	Catalog #
S 	9"-12" 23-30 cm	16"-32" 41-81 cm	Small Regular  29" - 33" 74-84 cm	111625
			Small Long  33" - 38" 84-97 cm	111626
M 	12"-15" 30-38 cm	16"-32" 41-81 cm	Medium Regular  29" - 33" 74-84 cm	111627
			Medium Long  33" - 38" 84-97 cm	111628
L 	15"-18" 38-46 cm	16"-32" 41-81 cm	Large Regular  29" - 33" 74-84 cm	111629
			Large Long  33" - 38" 84-97 cm	111630

Packaging Information

- Knee High – 12 pair / box; 6 measuring tapes in each dispenser box
- Thigh High – 6 pair / box; 3 measuring tapes in each dispenser box
- Waist High – 6 pair / box; 3 measuring tapes in each dispenser box

NOTE: Please refer to the product label and/or package insert for full instructions on safe use of these products.



BSN medical Inc. 5825 Carnegie Blvd. Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325 www.jobst.com
To order toll-free: JOBST (+1) 800 537 1063

50153 R12

© 2008 BSN medical Inc.



Comfort, Health and Style!

Printed in U.S.A.

REV 02/08